



REBOTEC® Rehabilitationsmittel GmbH
 DE-49610 Quakenbrück, Artlandstr. 57-59
 Telephone: +49 (0) 5431/9416-
 Fax: +49 (0) 5431/9416-66
 Web: www.rebotec.de
 Email: info@rebotec.de



BA-Nr.: 02.07.085/2022-07-A01

User Manual

Shower and toilet wheelchair for children

	Augsburg 3 inches	Augsburg 4 inches	Augsburg 5 inches
REF	339.03.97	339.04.97	339.05.97
	80 kg	80 kg	80 kg



Figure similar



Read this User Manual **before using the device for the first time** and keep it at hand for the user as well as for future use and subsequent sales.

**Table of contents**

1	Product description	3	12	Troubleshooting.....	8
1.1	Introduction	3	13	Care and maintenance.....	9
2	Features	3	13.1	Cleaning and care	9
2.1	Intended purpose	3	13.2	Maintenance	9
2.2	Conditions of use	3	14	Spare parts & accessories	10
2.3	Storage & transport.....	4	15	Drawing & dimensions	11
2.4	Packaging	4			
2.5	Disposal	4			
3	Symbols & nameplates	4			
4	Safety instructions	4			
5	Incident reporting.....	5			
6	Warranty.....	5			
6.1	Complaint.....	5			
7	Reuse, lifetime	5			
8	Scope of delivery.....	5			
9	Setting up for use.....	6			
9.1	Folding down the footrests	6			
9.2	Adjusting the footrest height	6			
9.4	Attachment of height adapter for footrest	6			
9.5	Assembling the backrest.....	6			
9.6	Mounting the backrest cushion	6			
10	Product data	7			
11	Operation of the product	7			
11.1	Sitting down.....	7			
11.2	Getting up	7			
11.3	Swivelling the armrests	8			
11.4	Parking brake operation	8			
11.5	Removing the footrests	8			
11.6	Handling of the toilet bucket	8			
11.7	Removing the shower/ toilet seat	8			
11.8	Height adjustment of the backrest cushion	8			

1 Product description

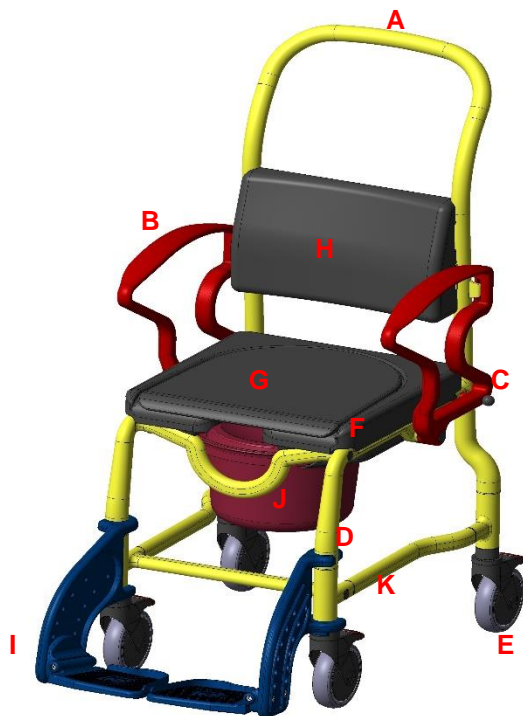


Figure shows:

A	Backrest
B	Armrest (swivelling)
C	Locking bolts
D	Chair frame with struts
E	Wheel insert with parking brake
F	PUR seat, hygiene opening
G	PUR seat cushion (removable)
H	PUR backrest cushion
I	Foot support with footrest (adjustable)
J	Bucket holder
K	Nameplate

1.1 Introduction

- The product described here is a medical device of risk class I according to **(EU) 2017/745 MDR**.
- In this user manual, you will find information on the safe and proper initial operation, use, maintenance and care of this product.
- Carefully read this user manual before using the device. Follow the safety instructions and act accordingly.

2 Features

2.1 Intended purpose

- The product is intended for the care of patients with limited ability to stand and/or walk. It ensures showering while sitting and going to the toilet, even if a conventional toilet or bathroom is out of reach. If designed appropriately, it can be slid over a conventional toilet. The patient must have sufficient sitting ability and upper body musculature. If necessary, a nurse must be present during use. The product is not intended for prolonged sitting and transfer.

Specification

- For children
- Maximum user weight: 80 kg
- Height adjustable footrests
- Wheelchair, 4 swivel castors, lockable
- Indoor use

Indication

- If it is impossible for patients to visit a stationary toilet, due to damage to the function of the lower extremities with limited or eliminated mobility and muscle strength and/or damage to the vestibular functions or tendency to fall due to other causes, while at the same time maintaining a sufficiently stable thorax posture for sitting and sufficient residual functions of the upper extremities.
- The shower and toilet wheelchairs are intended solely to assist in showering or bathing, toileting or transferring a person indoors.
- Medical condition of the patient:
- The use of a chair by patients with cognitive disorders (reduced mental, physical, sensory or intellectual abilities) must be decided in advance by a medical decision (prescription of the product) or on site by the nursing personnel.

2.2 Conditions of use

- Temperature range: 5°C to 35°C; direct radiant heat on the product must be avoided.
- Relative humidity: 40%-75%. Outside of these conditions, the product must only be used for a short time.
- The product is not suitable for use in swimming pools or saunas, as surface discolouration and limitations of mechanical strength may occur.
- Disassembly for transport or storage is possible.

- There must be a flat surface of sufficient size for the hoist to be placed on. See 15.

2.3 Storage & transport

- Temperature range: 5 °C to 25 °C; direct radiant heat on the product must be avoided.
- Store the product in dry rooms with a humidity of up to 65%.
- The shower should be designed so that there is a flat surface of sufficient size.

2.4 Packaging

- We recommend you keep the packaging for a later transport.
- Dimensions: 560mm x560mm x640mm

2.5 Disposal

- Disposal and recycling of used products and packaging must be carried out in accordance with currently valid regulations. Please contact a disposal company for further information.
- The Infection Protection Act must be observed.
- Please observe any labels and information on packaging material and act accordingly.

3 Symbols & nameplates

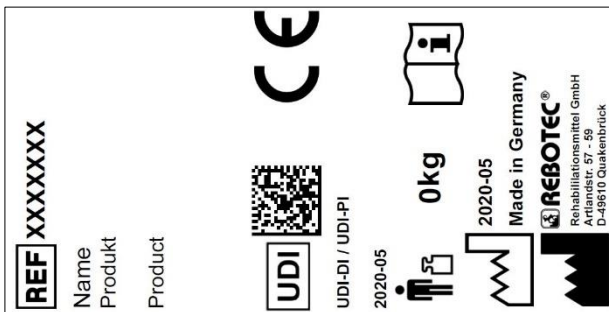


Figure similar.



The nameplate must not be removed!

For ordering spare parts and processing complaints, the device name, serial number and year of manufacture will be required.

Symbol	Meaning	Symbol	Meaning
	Read the manual!		Year of manufacture of Device
SN	Serial number		<i>Important information!</i>
REF	Article number		<i>Maximum user weight in kg</i>

Symbol	Meaning	Symbol	Meaning
	Care instructions		Maintenance instructions
	Applicable for indoor use only.		Manufacturer
	Visual signal		Language
	Read the User Manual!		Do not dispose of as household waste.
CE	Product complies with (EU) 2017/745 MDR		
	Caution! Observe the safety instructions		
UDI	Unique Device Identifier (Data: GTIN / LOT (SN) / year of manufacture)		
MD	Medical device label This product is a medical device.		

4 Safety instructions



Please observe the following safety instructions:

- Before using the device for the first time, the User Manual must be read completely in order to avoid hazards or damage caused by maloperation.
- Before each use, check the device for visible damage or deficiencies.
- In case of malfunctions or damage of any kind, the device must not be used.

Caution: Danger of falling!

- The chair must only be used with the armrests locked in place when sitting and moving about.
- Sitting down, getting up, transferring of a patient should be done under supervision or with the help of a trained caregiver, depending on the patient's state of health.
- Patients who are prone to uncontrolled actions or movements must be supervised when using the chair.
- The chair must **only** be moved on a slope or incline with the help of a caregiver. Generally, on an uphill slope the chair should be pushed forward and on a downhill slope it should be pulled backward.

Caution: Danger of breakage!



- The product is only suitable for the intended use. Observe the permitted user weight.
- Never carry or lift the product while a patient is sitting on it.
- Do not pull or hold on to additionally attached components or accessories.

Caution: Danger of tipping over!

- The chair must only be used with the armrests locked in place when sitting and moving about.
- The chair must only be pushed by the push handle (backrest).
- **Only** move the chair at a moderate walking pace (with and without patients).
- Do not lean out of the chair (both in standstill and while moving). If the swivel castors are in an unfavourable position (pointing inwards), stability of the chair is limited.

Caution: Danger of pinching or catching!

- When folding the armrests up or down, do not reach into the joint guide so that no parts of the body or other objects are trapped. To this end, also see 9 on page 6.
- When attaching the seat plate, do not reach between the seat surface and the seat frame so that no body parts are trapped.
- When pulling out and pushing in the toilet bucket, there is a risk of trapping soft tissue of the patient. To prevent injuries, the bucket may only be pulled out or pushed in when no user is sitting in the chair.

Caution: Risk of injury!

- There is a risk of injury with surface temperatures > 40°C. Therefore, do not expose the product to high temperatures (e.g. sunlight, radiators, hot water). Allow the product to cool down before use.



If the product is used by patients with insensitive skin (no temperature perception or skin damage), a heat check (e.g. touching with the back of the hand) must be carried out by the nursing staff.

5 Incident reporting

- Serious incidents in connection with the product must be reported immediately to the manufacturer and BfArM.
 - BfArM: www.bfarm.de/DE
 - Use the reporting functions and provided forms.
 - Manufacturer: pms@rebotec.de

6 Warranty

- For this product the manufacturer grants a warranty of 12 months. Compliance with the general terms and conditions (www.rebotec.de/agbs), as well as the intended use, are a basic prerequisite. The statutory warranty provisions shall apply.
- Unauthorised modifications to this product will void the product's conformity and the warranty.

6.1 Complaint

- Please contact us before returning the product.
- To reduce transport damage, use the original packaging if possible.
- The Infection Protection Act must be complied with.
 - Please make sure that the product does not pose a risk of infection when returned.
 - Enclose the information sheet on safety with the product.
 - Costs may incur for returns that are not marked non-objectionable.

7 Reuse, lifetime

- The product is suitable for reuse. How often the product can be reused depends on the frequency and nature of use.
- Before reuse, the product must be reprocessed hygienically and the technical and functional safety and reliability of the product must be checked and restored if necessary. See 13.
 - For more detailed information, please see the reprocessing instructions (02.12.156) and the maintenance instructions (02.11.150).
 - The documents can be downloaded from the following link.
 - www.rebotec.de/downloads
- If the product is used as intended, its lifetime is 5 years. The effective lifetime may vary depending on the frequency and intensity of use and the general condition.

8 Scope of delivery

- 1x shower / toilet wheelchair
 - 1x toilet bucket
 - 2x Height adapters
- 1x user manual (02.07.085)
- 1 Allen key


- The reprocessing and maintenance instructions can be ordered or viewed and downloaded at www.rebotec.de/downloads.

9 Setting up for use

- The product has been carefully inspected in the factory for freedom from defects and completeness. Please check the product for any possible damage which may have occurred during transport and for completeness of the scope of delivery upon receipt. Assembly or adjustment shall be carried out by the specialist dealer


9.1 Folding down the footrests

- Turn (1) the footrests forwards until they engage automatically. Fold (2) the footrests down. Adjust the height of the footrests if necessary.

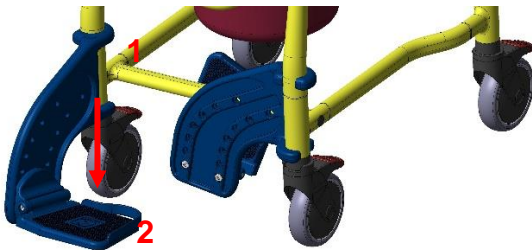

9.2 Adjusting the footrest height


- The footrest height can be adjusted in 7 positions.

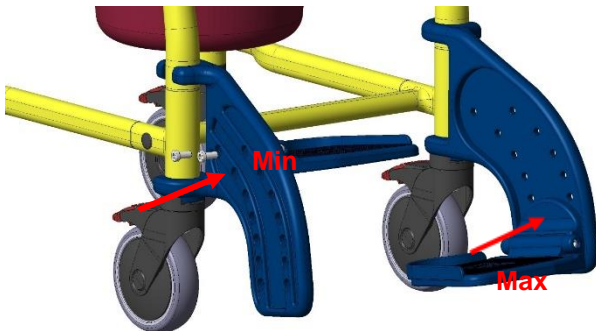


Loosen and remove the screws on the side of the footrest using an Allen key (SW 5). Select the appropriate height level (1-7) and screw the footrest back in place. Make sure that the screws are firmly tightened!

9.4 Attachment of height adapter for footrest

- To reduce the space between footrest and seat even more, an additional height adapter can be attached between the front wheel insert and the swivel arm of the footrest.

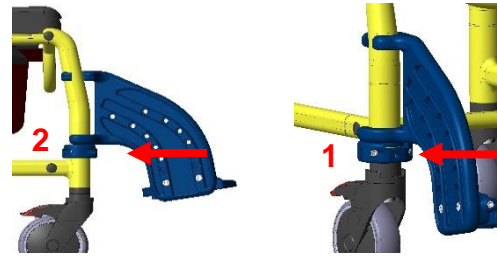

- The height adapter can be installed either below (1) or just above (2) the frame strut.

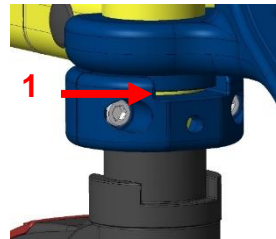

- When installing the adapter, make sure that the recess (1) faces upwards so that the footrest can latch in properly.

- With the Augsburg chair with 3" wheels, the two height adapters for the footrest are pre-installed to avoid floor contact with the footrest set to maximum height. With other variants, both height adapters can be installed as required.


9.5 Assembling the backrest



- Swivel the armrests backwards, place the backrest on the tubes of the chair frame and push it onto the base frame as far as it will go.
- Swivel the armrests forwards again so that they engage in the groove on the backrest and with the locking bolt in the base frame.

9.6 Mounting the backrest cushion

- The backrest cushion is attached to the cross strut of the backrest. (1&2)

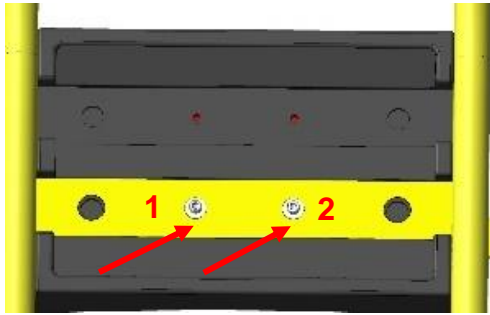

10 Product data

- We reserve the right to make technical developments and improvements with respect to the design shown in this User Manual.
- See 15.

Characteristic		Value
Augsburg 3 inches		
1.	User weight	80 kg
2.	Product weight*	16.4 kg
3.	Total weight*	19.0 kg
Skin compatibility		DIN EN 10993-1/-5
Characteristic		Value
Augsburg 4 inches		
1.	User weight	80 kg
2.	Product weight*	16.8 kg
3.	Total weight*	19.4 kg
Skin compatibility		DIN EN 10993-1/-5
Characteristic		Value
Augsburg 5 inches		
1.	User weight	80 kg
2.	Product weight*	17.2 kg
3.	Total weight*	19.6 kg
Skin compatibility		DIN EN 10993-1/-5

Without accessories

11 Operation of the product

- Before use, it must be ensured that the product is in proper condition. If defects or malfunctions are evident, the product must not be used. Have the defects corrected by a specialist dealer before use.



If necessary, depending on the patient's limitations (adequate mobility and muscle strength), sitting down and getting up must be done with the help of a caregiver. Subsequent adjustments may only be made by qualified personnel.

11.1 Sitting down

- Before sitting down and standing up, make sure that the product is securely positioned to prevent it from slipping or tipping over, that at least two wheels are braked, that the armrests are properly

engaged, that the seat surfaces are firmly in place, and that the footrests are folded to the side or up.

1.



- The user must position themselves as close as possible to the product. If necessary, they can orientate or support themselves with their hands on the armrests.

- If this is not possible due to their physical limitations, the user must seek the assistance of a competent person. This can be with the help of an assistive device



Attention: Danger of falling!

2.




- Falling down or slumping on the chair seat must be avoided. The patient should sit on the full chair surface, if possible, and not only on the edge of the seat

- When transporting people, the entire surface of the backrest and the footrests must be used if possible.

11.2 Getting up

- Before standing up, make sure that the product is securely positioned to prevent it from slipping or tipping over, that at **least two wheels** are braked, that the armrests are properly engaged, that the seat surfaces are firmly in place, and that the footrests are folded to the side or up.
- The patient must move forward on the chair so that their feet are on the floor.
- The patient must support themselves with their arms on the armrests and slowly rise from the seat.

3.

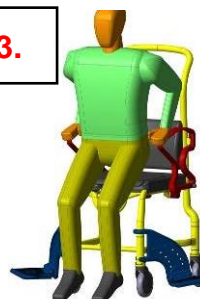

- Do not lean your body too far forward to avoid a potential risk of falling.

- The patient should be helped to stand up by the nursing staff



Caution: Danger of falling!

4.




- The patient must never leave their feet on the footrests when sitting down or getting up.

11.3 Swivelling the armrests

- Pulling out the locking bolts (A) releases the locking mechanism and the armrests can be swivelled backwards.
- When folding up the armrest, check that the catch on the backrest (B) and the locking bolt (A) are seated correctly.

Caution: Danger of pinching or catching!


- The armrests can be swivelled with both hands. To do this, keep the locking bolt (A) pulled during the entire swivelling process and guide the armrest to point (C) with the other hand.
- Only touch the outer area and do not reach into or through the armrest.

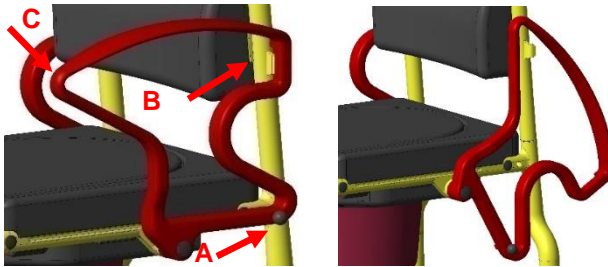

Check the locking at regular intervals, as possible limescale deposits on the locking mechanism can impair function and smooth operation.

11.4 Parking brake operation

- With your foot, press down the parking brake lever (1). The wheel is now braked and fixed in its direction. The wheel is released by lifting the actuating lever with the foot (2).

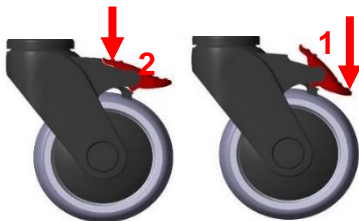

11.5 Removing the footrests

- Remove or fold up the seat. Then, raise the footrest by approx. 100 mm and turn outwards by 90 degrees.
- Now you can push the footrest all the way up and pull it out

11.6 Handling of the toilet bucket


- From the back of the chair, slide the toilet bucket under the seat. To remove, slightly lift the bucket and then pull it out backwards.
- In the version with a 2-section toilet seat, you can alternatively insert or remove the toilet bucket from above by lifting the seat.

11.7 Removing the shower/ toilet seat

- The seat can be removed from its locked position on the chair frame by lifting it off for cleaning or replacement.

11.8 Height adjustment of the backrest cushion

- The backrest cushion is attached to the cross strut of the backrest. It can be adjusted in two fixed heights. By loosening the screws, the backrest cushion can be adjusted by 80 mm.

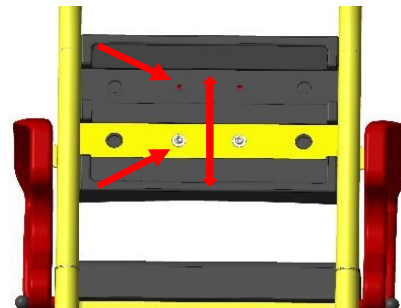

12 Troubleshooting

Fault	Potential cause	Remedy
Armrest cannot be swivelled down	Deposits in the locking device (locking bolt)	Cleaning, checking regularly
Locking bolt cannot be pulled out	Deposits, tension spring jammed or defective	Cleaning, Replace the locking bolt
Footrest cannot be folded down	Deposits in the joint, Bracing	Cleaning, Check screw connection
Wheel lock does not brake	General defect	see maintenance!

Fault	Potential cause	Remedy
Turning the wheel is stiff	Deposits	Cleaning
Wheel does not turn	General defect	see maintenance!
Uneven running of the wheel	Severe uneven tread wear	see maintenance!
Chair frame too unstable	Screw connections too loose	Screw connections: Tighten frame, wheel inserts
Wheel insert bent/distorted	Hitting an obstacle	see maintenance!

13 Care and maintenance

13.1 Cleaning and care



The product can be cleaned using a mild conventional cleaning and disinfecting agent. After cleaning, wipe the product with a clean, moist cloth and then wipe it dry with a dry cloth.

- Do not use scouring agents or cleaning agents with aggressive ingredients (acids, thinners), as well as hard cleaning objects (brushes), as otherwise the surfaces may be damaged or deposits may form on the materials.



Keep the product clean and clean it regularly. Special attention must be paid to hygiene in the sensitive areas.

- The product is suitable for manual cleaning and disinfection. Please observe the reprocessing instructions (02.12.156).

13.2 Maintenance

- Wear and tear that is not recognised in time and/or improper handling, as well as irregular or no maintenance may result in accidents.



*For safety reasons, the product must be professionally maintained at **least once a year**.*

- Check product for visible damage
 - (cracks, breakage, loose parts)
- Check stability before use

Maintenance/inspection scope

- Maintenance should include the following inspection work and should be documented.
- All bolted connections for tightness (especially the wheel inserts and the frame)

- Function of the armrests and locking bolts
- Function of the footrest (replace defective footrests)
- Running characteristics and tread of the wheels (replace defective wheels)
- Function of the parking brakes (replace defective brakes)
- Inspect accessories for deficiencies

Repair

- If you have any questions concerning the maintenance of the chair, please contact an authorised dealer or a medical and healthcare supplies store.
- In the event of damage and/or repairs, contact your authorised dealer immediately.
- **Never** make **any** unauthorised modifications or conversions to the chair. These will only adversely affect the safety and functioning of the chair.
- Only use OEM spare parts by the manufacturer REBOTEC for repairs. Observe all guarantee and warranty information.
- After maintenance or repair work has been carried out we recommend you clean the device for hygienic reasons before using it again.
- Follow all the instructions for ordering spare parts, reuse and the lifetime.



For documenting the maintenance and repair work, REBOTEC provides a Maintenance Report. *The operating instructions and maintenance log can be ordered or viewed via www.rebotec.de. It is also possible to print the information and read it when needed.*

14 Spare parts & accessories




No.	Art. no.	Designation
1	301.44	Toilet bucket, standard
2	309.10	Bucket holder
3	440.10.01	PUR shower seat padding
4	446.00.81	Wheel with brakes, for toilet chairs
5	450.10.05	Armrest engagement lock, complete
6a	447.XX.57	Footrest complete, left
6b	447.XX.59	Footrest complete, right
7	445.10.02	PUR backrest cushion
-	444.00.97	Adjustment of the seat width & depth
-	444.03.39	Headrest unit
-	440.54.67	Adapter for height adjustment of the legrests
-	02.07.085	User Manual
-	02.11.150	Maintenance log
-	02.12.156	Reprocessing Instructions

XX= Colour variant



Use only REBOTEC spare parts and accessories to ensure the proper and safe functioning of the device. Subject to change without notice.

15 Drawing & dimensions

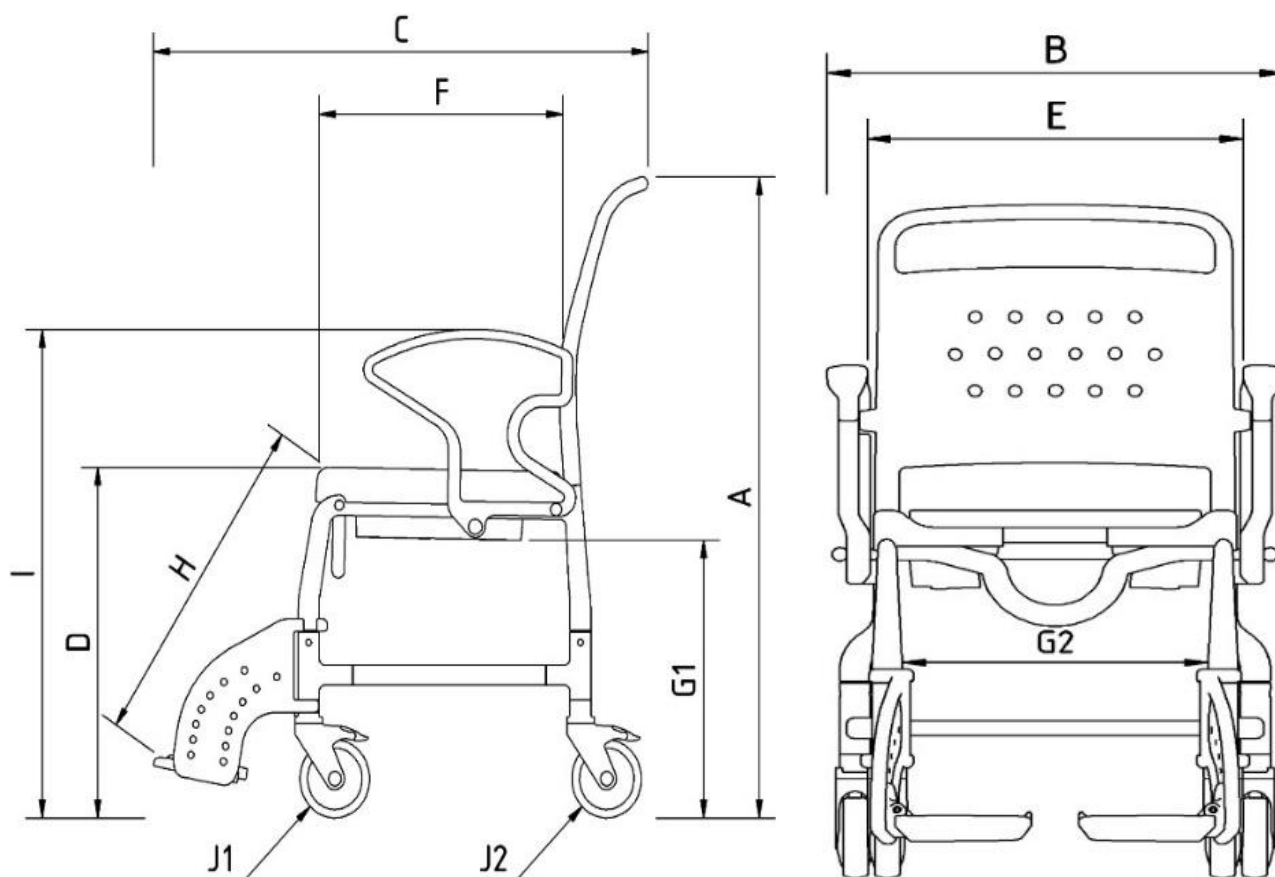

No.	Designation	3" wheel dimensions / mm	4" wheel dimensions / mm	5" wheel dimensions / mm
A	Height	1080	1110	1130
B	Width	520	520	520
C	Depth	830	830	830
D	Seat height	510	540	560
E	Seat width	420	420	420
F	Seat depth	350	350	350
G1	Slide-over height	380	410	430
G2	Slide-over height	330	330	330
H*	Knee height of footrest max.	380-540	380-540	380-540
I	Armrest height	720	750	770
J1	Front castor diameter	75	100	125
J2	Rear castor diameter	75	100	125

*Take into account the height adapter



Dimensions and weights may vary depending on the equipment. There might be minor deviations due to production tolerances. Subject to change without notice.

EN



REBOTEC® Rehabilitationsmittel GmbH
D-49610 Quakenbrück, Artlandstr. 57-59
Telephone: +49 (0) 5431/9416-
Fax: +49 (0) 5431/9416-66
Web: www.rebotec.de
Email: info@rebotec.de



Read this User Manual **before using the device for the first time** and keep it at hand for the user as well as for future use and subsequent sales.