



EN



REBOTEC® Rehabilitationsmittel GmbH
 DE-49610 Quakenbrück, Artlandstr. 57- 59
 Telephone: +49 (0) 5431/9416-
 Fax: +49 (0) 5431/9416- 66
 Web: www.rebotec.de
 Email: info@rebotec.de



BA-Nr.: 02.04.008/2021-08-A01

User Manual

Walking frame

Walk-On / Type XS		Walk-On / Type S	Walk-On / Type M	Walk-On / Type L
REF	174.10	174.20	174.30	174.40
	150 kg	150 kg	150 kg	150 kg
Walk-On / Type XS with 2 castors		Walk-On / Type S with 2 castors	Walk-On / Type M with 2 castors	Walk-On / Type L with 2 castors
REF	175.10	175.20	175.30	175.40
	120 kg	120 kg	120 kg	120 kg



Figure similar



Read this User Manual **before using the device for the first time** and keep it at hand for the user as well as for future use and subsequent sales.

**Table of contents**

1. Product description	3
1.1. Introduction	3
2. Features	3
2.1. Intended purpose	3
2.2. Conditions of use	3
2.3. Storage & transport.....	3
2.4. Equipment & functions	4
2.5. Packaging	4
2.6. Disposal	4
3. Symbols & nameplates	4
4. Safety instructions	4
5. Incident reporting.....	5
6. Warranty.....	5
6.1. Complaint.....	5
7. Reuse, lifetime	5
8. Scope of delivery.....	6
9. Setting up for use.....	6
9.1. Unfolding.....	6
9.2. Adjusting the handle height.....	6
10. Product data	7
11. Operation of the product	7
11.1. Setting up for use.....	7
11.2. Use of the product.....	7
11.3. Braking system.....	7
11.4. Folding of the product	7
11.5. Replacing the brakes	8
12. Troubleshooting	8
13. Care and maintenance	8
13.1. Cleaning and care	8
13.2. Maintenance.....	8
-(cracks, breakage, loose parts).....	8
14. Spare parts & accessories	10
15. Drawing & dimensions	11

1. Product description



Figure shows:

A	Front bar
B	Rear bar
C	Handlebar
D	Handles
E1	Castor with parking brake
E2	Foot cap
F	Front leg
G	Rear leg
H	Star knob nuts, height adjustment
I	Folding mechanism
J	Nameplate

1.1. Introduction

- The product described here is a medical device of risk class I according to **(EU) 2017/745 MDR**.
- In this user manual, you will find information on the safe and proper initial operation, use, maintenance and care of this product.
- Carefully read this user manual before using the device. Follow the safety instructions and act accordingly.

2. Features

2.1. Intended purpose

- The product is used for support and relief in case of restrictions of the lower extremities. It allows the user to pursue their desire for mobility. There must

be sufficient upper body musculature for safe use. Persons with sensory impairments may not be able to use this item. The article does not impose any special requirements on the education of the user.

Specification:

REF	Designation	
174.10	Walk-On / Type XS	150kg
174.20	Walk-On / Type XS	150kg
174.30	Walk-On / Type XS	150kg
174.40	Walk-On / Type XS	150kg
175.10	Walk-On / Type XS with 2 castors	120kg
175.20	Walk-On / Type XS with 2 castors	120kg
175.30	Walk-On / Type XS with 2 castors	120kg
175.40	Walk-On / Type XS with 2 castors	120kg

- Height adjustable handle
- Foldable
- Indoor use

Indication

- Impairment of mobility in the case of damage to locomotive/balance system with sufficiently preserved walking ability/coordination, if the use of other walking aids is insufficient
- - To maintain/promote/secure walking
- During use, care must be taken to ensure sufficiently spacious living conditions.

2.2. Conditions of use

- Temperature range: 5°C to 35°C; direct radiant heat on the product must be avoided.
- Relative humidity: 40% to 75%. Outside of these conditions, the product must only be used for a short time.
- Folding for transport or storage is possible.
- This product may not be used to transport heavy objects.
- There must be a flat surface of sufficient size for the stool to be placed on. See 15.

2.3. Storage & transport

- Temperature range: 5° C to 25° C; direct radiant heat on the product must be avoided.
- Store the product in dry rooms with a humidity of up to 65%.

2.4. Equipment & functions

- Foldable
- Handle height adjustable (see 9.2 & 15)
- 2 swivelling castors with parking brake (only art. no. 175...)

2.5. Packaging

- We recommend you keep the packaging for a later transport.
- Dimensions type XS: 700 mm x 470 mm x 650 mm
- Dimensions type S: 700 mm x 470 mm x 750 mm
- Dimensions type M: 700 mm x 470 mm x 750 mm
- Dimensions type L: 700 mm x 470 mm x 850 mm

2.6. Disposal

- Disposal and recycling of used products and packaging must be carried out in accordance with currently valid regulations. Please contact a disposal company for further information.
- The Infection Protection Act must be observed.
- Please observe any labels and information on packaging material and act accordingly.

3. Symbols & nameplates

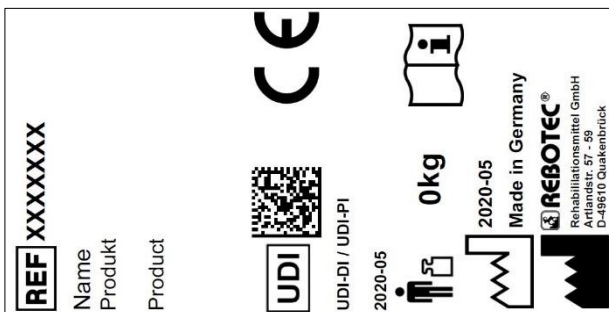


Figure similar.



The nameplate must not be removed!

For ordering spare parts and processing complaints, the device name, serial number and year of manufacture will be required.

Symbol	Meaning	Symbol	Meaning
	Read the manual!		Year of manufacture Device
SN	Serial number		Important information!
REF	Article number		Maximum user weight in kg

Symbol	Meaning	Symbol	Meaning
	Care instructions		Maintenance instructions
	Applicable for indoor use only.		Manufacturer
	Visual signal		Language
	Read the User Manual!		Do not dispose of household waste.
CE	Product complies with (EU) 2017/745 MDR		
	Caution! Observe the safety instructions		
UDI	Unique Device Identifier (Data: GTIN / LOT (SN) / year of manufacture)		
MD	Medical device label This product is a medical device.		

4. Safety instructions



Please observe the following safety instructions:

- Before using the device for the first time, the User Manual must be read completely in order to avoid damage caused by fault operation or hazards.
- Before each use, check the device for visible damage or deficiencies.
- In case of malfunctions or damage of any kind, the device must not be used.

Caution: Danger of falling!

- Be sure to check the following aspects before use:
 - Stability of the product.
 - Function of the parking brakes.
 - Proper locking of the folding mechanism.
 - Tight fit of the star knob nuts.
- Only use the product if completely assembled.
- Do not slump carelessly on the product.
- Only use the height adjustment in the holes provided for it.
- Avoid using the product on slippery or uneven ground.

- Make sure that the brakes are always in locked position before using the article. Only release the two brakes after you see you are able to stand stably.
- Do not start walking until you are standing upright and have grasped the handles with both hands.
- Always walk within the product frame and do not push it too far in front of you, otherwise there is an increased risk of falling due to incorrect loading.
- Users who are prone to uncontrolled actions or movements must be supervised when using the product.
- Use the article according to your mobility.

Caution: Danger of slipping!

- Do not grasp the handles with wet hands, as you could slip and lose your grip.

Caution: Danger of tipping over!

- Avoid walking on steep inclines or declines; otherwise you could lose control of the product.
- Do not use the product on stairs or steps.
- Do not hang any objects on the product.
- Do not lean too far over the handlebar.
- Avoid excessively leaning over the product.

Caution: Danger of breakage!

- Observe the permitted user weight (2.1).
- The product is not a means of transport for heavy objects.
- The product is not suitable for passenger transport.
- Do not expose the product to great temperature fluctuations (2.2 & 2.3).

Caution: Danger of pinching or catching!

- There is a risk of pinching or catching on all moving parts of the product. Be especially careful when unfolding and folding that no body parts or objects are caught between the moving parts.

Caution: Risk of injury!

- There is a risk of injury with surface temperatures > 40°C. Therefore, do not expose the product to high temperatures (e.g. sunlight, radiators, hot water). Allow the product to cool before use.



If the product is used by patients with insensitive skin (no temperature perception or skin damage), a heat check (e.g. touching with the back of the hand) must be carried out by the nursing staff.

5. Incident reporting

- Serious incidents in connection with the product must be reported immediately to the manufacturer and BfArM.
 - BfArM: www.bfarm.de/DE
 - Use the reporting functions and provided forms.
 - Manufacturer: pms@rebotec.de

6. Warranty

- For this product the manufacturer grants a warranty of 12 months. Compliance with the general terms and conditions (www.rebotec.de/agbs), as well as the intended use, are a basic prerequisite. The statutory warranty provisions shall apply.
- Unauthorised modifications to this product will void the product's conformity and the warranty.

6.1. Complaint

- Please contact us before returning the product.
- To reduce transport damage, use the original packaging if possible.
- The Infection Protection Act must be complied with.
 - Please make sure that the product does not pose a risk of infection when returned.
 - Enclose the supplied information sheet on safety with the product.
 - Costs may incur for returns that are not marked non-objectionable.

7. Reuse, lifetime

- The product is suitable for reuse. How often the product can be reused depends on the frequency and nature of use.
- Before reuse, the product must be reprocessed hygienically and the technical and functional safety and reliability of the product must be checked and restored if necessary. See 13.
 - For more detailed information, please see the reprocessing instructions (02.12.158) and the maintenance instructions (02.11.149).
 - The documents can be downloaded from the following link.
 - www.rebotec.de/downloads
- When used as intended, the product has a lifetime of 3 years. The effective lifetime may vary

depending on the frequency and intensity of use and the general condition.

8. Scope of delivery

- Walk-On
 - Walking frame
 - User manual (02.04.008)
- Walk-On with 2 castors
 - Walking frame with 2 swivelling castors with parking brake
 - User manual (02.04.008)
- The reprocessing and maintenance instructions can be ordered or viewed and downloaded at www.rebotec.de/downloads.

9. Setting up for use

- The product has been carefully inspected in the factory for freedom from defects and completeness. Please check the product for any possible damage which may have occurred during transport and for completeness of the scope of delivery upon receipt. Assembly or adjustment shall be carried out by the specialist dealer.

9.1. Unfolding

- Place the product on its rear legs and push the frame apart with the handle bar
- The folding mechanism locks automatically in its final position.



- Check that the item is fully unfolded and the folding lock is engaged.



9.2. Adjusting the handle height

- The height of the handlebar must be adjusted to your body height.
- Loosen the two star knob screws (1) and pull them out. Move the handlebar (2) to the desired height. The holes of the two tubes must be aligned on top of one another. Then tighten the star knob screw again.



- Check the function of the parking brakes. (See 11.3, 11.5 & 13.2)

10. Product data

- We reserve the right to make technical developments and improvements with respect to the design shown in this User Manual.
- See 15.

Characteristic		Value
Walk-On / Type XS		
1.	Product weight*	3.2 kg
2.	Total weight*	5.4 kg
Walk-On / Type S		
1.	Product weight*	3.4 kg
2.	Total weight*	5.2 kg
Walk-On / Type M		
1.	Product weight*	3.8 kg
2.	Total weight*	5.6 kg
Walk-On / Type L		
1.	Product weight*	3.8 kg
2.	Total weight*	6.0 kg
Walk-On / Type XS with 2 castors		
1.	Product weight*	3.6 kg
2.	Total weight*	5.4 kg
Walk-On / Type S with 2 castors		
1.	Product weight*	3.6 kg
2.	Total weight*	5.2 kg
Walk-On / Type M with 2 castors		
1.	Product weight*	4.0 kg
2.	Total weight*	5.6 kg
Walk-On / Type L with 2 castors		
1.	Product weight*	4.0 kg
2.	Total weight*	6.0 kg
Skin compatibility		DIN EN 10993-1/-5
Flammability**		DIN EN 1021-1/-2

*Without accessories.

**Flame retardant: For products made of PUR material.

11. Operation of the product

- Before use, it must be ensured that the product is in proper condition. If defects or malfunctions are evident, the product must not be used. Have the defects corrected by a specialist dealer before use.

11.1. Setting up for use

- The product must be unfolded and the latches engaged. Check that the parking brakes are securely in place and that they function properly (see: 9.1 & 11.3).

11.2. Use of the product

- Support yourself on the handles while walking.
- Always walk as upright as possible and inside the frame.
- Do not lean too far over the handlebar.
- Only use level paths and avoid obstacles such as curbs and doorsteps.
- Avoid slippery and loose floor coverings.
- While walking, both brakes must be released.

11.3. Braking system

Improper use of the brake can cause unsafe situations and damage (increased wear, limited function).

The product should not be used if the brakes are applied or the wheels are locked.

Parking brake

- Press down the brake levers of both front wheels with your foot until they engage. The two front wheels are now braked. To release the brake, pull the brake levers back up.



11.4. Folding of the product


Unfolding

- Place the product on its rear legs and push the frame apart with the handle bar.

- See 9.1.

Folding

- Pull the two locking bolts (1) upwards with one hand each until the lock is released. Then lift the product slightly and swing the bracket (2) upwards.

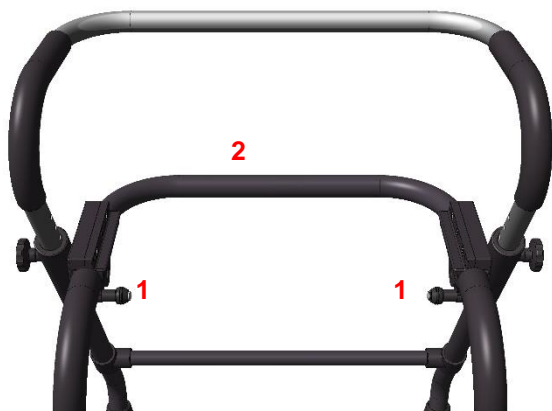

11.5. Replacing the brakes

- If the braking effect of the brakes begins to noticeably deteriorate, the product may no longer be put into operation and must be repaired by a specialist dealer.

12. Troubleshooting

Fault	Potential cause	Remedy
Wheels turn sluggishly	Deposits	Cleaning, checking regularly
Engagement bolts are jammed	Deposits	Cleaning, replace engagement bolts
No or reduced braking function	Brake worn, defective	Maintenance! Have castors replaced
Wheel set bent, broken	Hitting an obstacle	Maintenance! Replacement
Visible cracks or breaks	Various causes	Maintenance! Replacement
Slip resistance reduced	Worn foot capsules	Maintenance! Replacement
	Wetness	Dry foot capsules

13. Care and maintenance

13.1. Cleaning and care



The product can be cleaned using a mild conventional cleaning and disinfecting agent. After cleaning, wipe the product with a clean, moist cloth and then wipe it dry with a dry cloth.

- Do not use scouring agents or cleaning agents with aggressive ingredients (acids, thinners), as well as hard cleaning objects (brushes), as otherwise the surfaces may be damaged or deposits may form on the materials.
- The product is suitable for manual cleaning and disinfection. Please observe the reprocessing instructions (02.12.155).

13.2. Maintenance

- Wear and tear that is not recognised in time and / or improper handling, as well as irregular or no maintenance may result in accidents.



*For safety reasons, the product must be professionally maintained at **least once a year**.*

- Check product for visible damage
 - (cracks, breakage, loose parts)
- Check stability before use

Maintenance/inspection scope

- Maintenance should include the following inspection work and should be documented.
- Visual inspection for damage, wear and corrosion of all frame parts such as:
 - Frame
 - Handlebar
 - Foot capsules (replace in pairs)
 - Front wheels (replace in pairs)
- Check all connecting elements (screws, nuts) for completeness, fit and function
- Function of the folding mechanism
- Function of the height adjustment mechanism
- Function of the brakes (parking brake)
- In the event of damage and/or repairs, contact your authorised dealer immediately.
- Never attempt **any** unauthorised modifications or conversions to the product. These may impair the safety and proper functioning of the device. **In such cases, REBOTEC will not assume any liability.**
- Only use OEM spare parts for repairs. See 6.
- After maintenance or repair work has been carried out, we recommend you clean the product before using it again.
- Please note section 7& 14.



-
- Please observe the maintenance instructions (02.11.149).

14. Spare parts & accessories




No.	Art. no.	Designation
1	400.00.84	Wheel for Walk-On, black
2	169.10.30	Foot cap
4	853.06.40	Star knob, complete
-	280.10.48	Back rest pad
-	02.04.008	User Manual
-	02.11.149	Maintenance Instructions
-	02.12.155	Reprocessing Instructions



Use only REBOTEC spare parts and accessories to ensure the proper and safe functioning of the device.

15. Drawing & dimensions

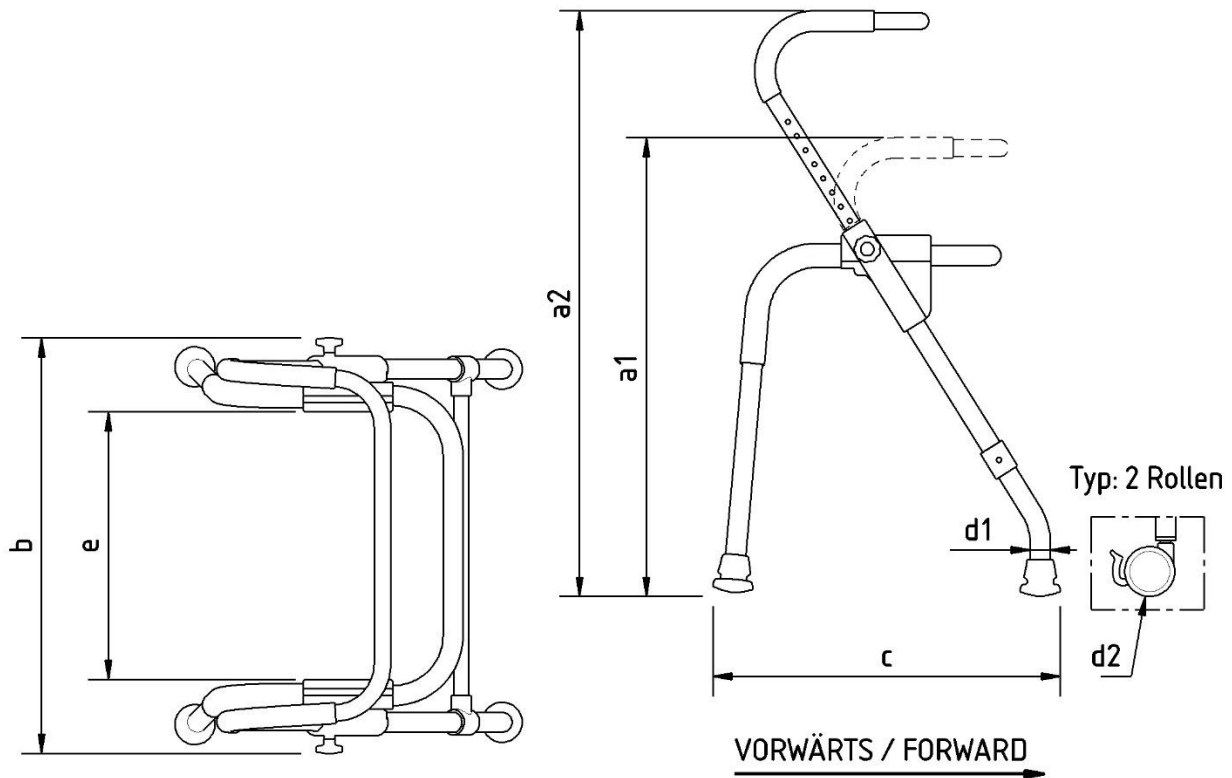

No.	Designation	Type XS / Dimensions in mm	Type S / Dimensions in mm	Type M / Dimensions in mm	Type L / Dimensions in mm
a1	Height min.	620	700	680	790
a2	Height max.	705	805	875	980
b	Width	565	565	625	625
c1	Length (standard)	440	440	510	510
c2	Length (with 2 castors)	410	410	460	460
d1	Inner diameter of foot capsule	30	30	30	30
d2	Front castor diameter	75	75	75	75
e	Inner width	340	340	400	400
bxc1	Floor space (standard)	565 x 440	565 x 440	625 x 510	670
bxc2	Floor space (with 2 castors)	565 x 410	565 x 410	625 x 460	625 x 460



Dimensions and weights may vary depending on the equipment. There might be minor deviations due to production tolerances. Subject to change without notice.



REBOTEC® Rehabilitationsmittel GmbH

D-49610 Quakenbrück, Artlandstr. 57- 59

Telephone: +49 (0) 5431/9416-

Fax: +49 (0) 5431/9416- 66

Web: www.rebotec.de

Email: info@rebotec.de



Read this User Manual **before using the device for the first time** and keep it at hand for the user as well as for future use and subsequent sales.